

# Ingredients

## Ingredients:

PVP, Water, PEG-8, Acrylates Copolymer, 11% Hydrogen Peroxide, Sodium Hydroxide, Sodium Saccharin.

## CONTRAINDICATIONS:

Do not use this light device if you:

- Have oral cancer
- Are being treated with photodynamic therapy by your doctor
- Are taking light sensitive medicines like aminolevulinic acid or porphyrin containing medications
- Or have sensitivity to light

## GENERAL WARNINGS AND PRECAUTIONS:

This is not a toy. Keep the device and the Whitestrips out of reach of children and pets. Not recommended for use by persons under the age of 18

Do not swallow the plastic strip.

Avoid eye contact. Rinse eyes thoroughly with water if gel comes in contact with them. Handle strips with dry hands.

Discontinue use of the device if it is damaged or behaves in an unexpected manner.

Keep the device and the Whitestrips away from high temperature environments (direct sunlight, radiator, heater) when storing.

Gel that comes into contact with the skin can cause temporary skin irritation and/or whitening. Wash hands with soap and water after applying the strips.

Do not eat, smoke, sleep, or drink (with the exception of water) while wearing Whitestrips.

Do not apply to irritated gums. If irritation develops discontinue use.

The Crest LED Accelerator Light is not intended for use by children or persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge, unless they are given supervision by a person for their safety. In general, we recommend that you keep the device out of reach of children. Children should be to ensure that they do not play with the device.

## DEVICE LIFETIME AND DISPOSAL:

Discard Crest LED Accelerator Light after last Whitestrip is used.

Batteries are included. No replacement necessary.

There are no user serviceable parts inside.

The device contains primary lithium batteries that are not rechargeable. Please dispose of the device according to local regulations.

Do not dispose of device in fire or incinerator.

## CLEANING AND STORAGE:

Rinse the Crest LED Accelerator Light mouthpiece with clean water and allow to air dry. Device is designed to be water resistant for cleaning in the sink.

Clean the device after each use.

Clean device only in recommended manner. Use of alternative methods may damage device.

Store Crest 3DWhitening Kits in a clean, dry, place at room temperature.

Keep The Crest LED Accelerator Light and Crest 3DWhitestrips out of reach of children and pets.

Avoid staring at the light. You might see temporary spots or after images. They are not harmful and will go away over time.

## Safety Warning:

Important usage information: Do not use this light device if you have oral cancer, are being treated with photodynamic therapy by your doctor, are taking light sensitive medicines like aminolevulinic acid or porphyrin medications, or have sensitivity to light. Keep out of reach of young children. Product is not intended for use in under the age of 18. Do not swallow plastic strip. Do not eat, sleep, smoke or drink (with the exception of while wearing Crest 3DWhitestrips. Your dentist and Crest want to keep your teeth healthy. Visit your dentist Crest 3DWhitestrips is not a substitute for regular dental care. Avoid eye contact. Rinse eyes thoroughly with if gel comes in contact with them. Handle strips with dry hands. Teeth whitening gel that comes into contact