

# Ingredients

## Ingredients:

Petrolatum, Water, Hydrogen Peroxide, Flavor, Sucralose, Sorbitan Palmitate

## Important Usage Information:

AVOID CONTACT WITH CLOTH, LEATHER OR FABRIC.

FOR FASTER TEETH WHITENING RESULTS USE UP TO 4 TIMES PER DAY. IF YOU EXPERIENCE DISCOMFORT, REDUCE USAGE.

THIS UNIQUE PRODUCT IS DESIGNED TO MINIMIZE TEETH WHITENING RELATED SENSITIVITY. IN THE UNLIKELY CASE THAT YOU DO EXPERIENCE SENSITIVITY, THE EFFECTS GENERALLY MILD AND WILL GO AWAY ONCE YOU STOP OR REDUCE USAGE.

CREST WHITENING EMULSIONS CONTACT WITH SKIN CAN CAUSE IRRITATION AND/OR WHITENING. WASH HANDS WITH SOAP AND WATER AFTER APPLICATION.

THIS PRODUCT IS NOT A SUBSTITUTE FOR REGULAR DENTAL CARE. VISIT YOUR DENTIST REGULARLY. IF YOU ARE IN NEED OF DENTAL WORK, HAVE DENTAL WORK, OR STAINING MEDICATION ASK YOUR DENTIST IF TEETH WHITENING IS APPROPRIATE. THIS PRODUCT ONLY WHITEN NATURAL TEETH; IT WILL NOT WHITEN CAPS, CROWNS, VENEERS, FILLINGS DENTURES.

USE UP TO FIVE BUILT-IN APPLICATOR TUBES OR TWO WAND APPLICATOR TUBES PER YEAR.

THE CREST LED ACCELERATOR LIGHT WHITENS TEETH WHEN USED WITH CREST WHITENING EMULSIONS.

## Safety Warning:

CAUTION: FOR AGES 12 AND UP. KEEP OUT OF REACH OF YOUNG CHILDREN. THE CREST LED ACCELERATOR LIGHT IS NOT INTENDED FOR USE BY CHILDREN OR PERSONS WITH REDUCED SENSORY OR MENTAL CAPABILITIES OR LACK OF EXPERIENCE AND KNOWLEDGE, UNLESS ARE GIVEN SUPERVISION BY A PERSON RESPONSIBLE FOR THEIR SAFETY. IN GENERAL, WE THAT YOU KEEP THE DEVICE OUT OF REACH OF CHILDREN. CHILDREN SHOULD BE TO ENSURE THAT THEY DO NOT PLAY WITH THE DEVICE.

KEEP OUT OF EYES. IN CASE OF CONTACT WITH EYES, FLUSH THOROUGHLY WITH WATER. STORE IN A COOL PLACE. AVOID EXCESSIVE HEAT.

DO NOT USE THE CREST LED ACCELERATOR LIGHT DEVICE IF YOU HAVE ORAL CANCER, ARE BEING TREATED WITH PHOTODYNAMIC THERAPY BY YOUR DOCTOR, ARE TAKING LIGHT MEDICINES, LIKE AMINOLEVULINIC ACID, OR PORPHYRIN-CONTAINING MEDICATIONS, OR SENSITIVITY TO LIGHT.

WARNING: CAP CHOKING HAZARD – SMALL PART. KEEP OUT OF REACH OF CHILDREN UNDER THE AGE OF 3.

DISCONTINUE USE OF THE DEVICE IF IT IS DAMAGED OR BEHAVES IN AN UNEXPECTED MANNER.